

STOP THE VICIOUS CIRCLE OF DRY EYE



**For the effective and safe treatment of dry eye:
0.2 % hyaluronic acid with 0.001 % hydrocortisone**



PRODUCT INFORMATION


IDROFLOG[®] stops the vicious circle of dry eye

The vicious circle of dry eye


According to TFOS DEWS II¹, dry eye is a multifactorial disease of the ocular surface characterized by a loss of homeostasis of the tear film, and accompanied by ocular symptoms, in which tear film instability and hyperosmolarity, ocular surface inflammation and damage, and neurosensory abnormalities play etiological roles.

IDROFLOG[®]: 0.2% hyaluronic acid with 0.001% hydrocortisone

For the effective and safe treatment of dry eye

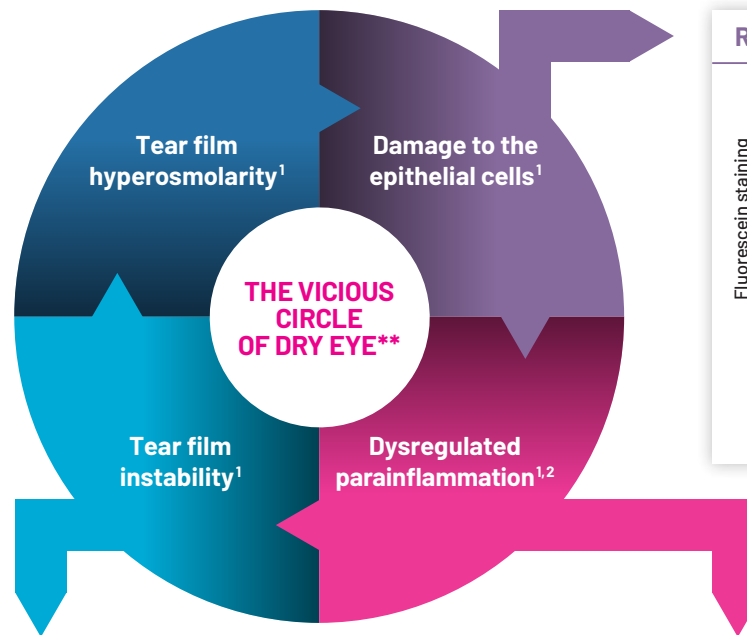


Reduction of corneal damage

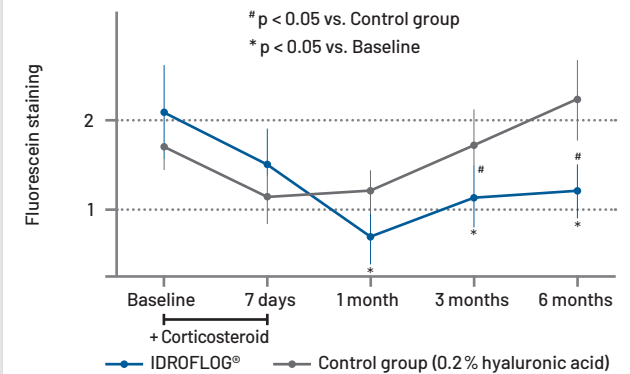


IDROFLOG[®] significantly reduces damage to the surface of the eye. Fig. modified after: Rolando M et al.³

Improvement of tear film stability

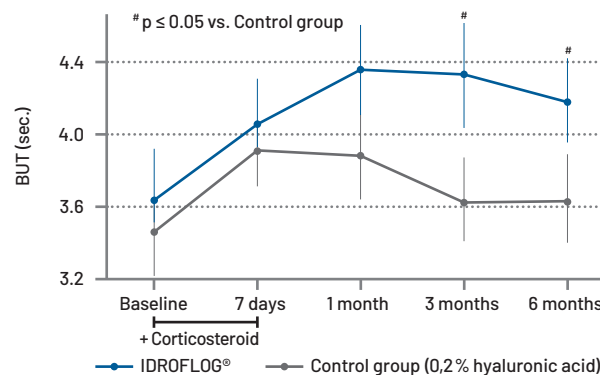


IDROFLOG[®] significantly improves the stability of the tear film. Fig. modified according to: Rolando M et al.³

Reduction in the number of infiltrating macrophages

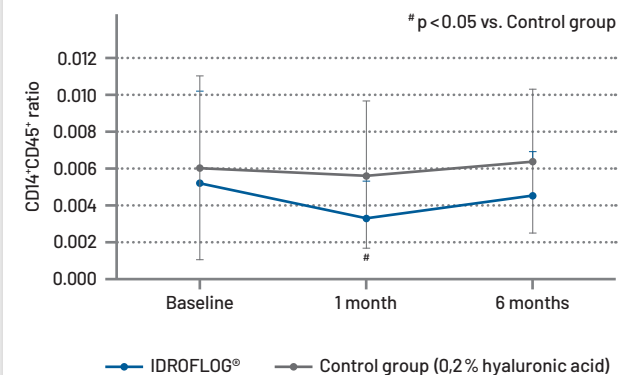

IDROFLOG[®] reduces the number of infiltrating macrophages. Fig. modified according to: Rolando M et al.³

IDROFLOG® safety profile

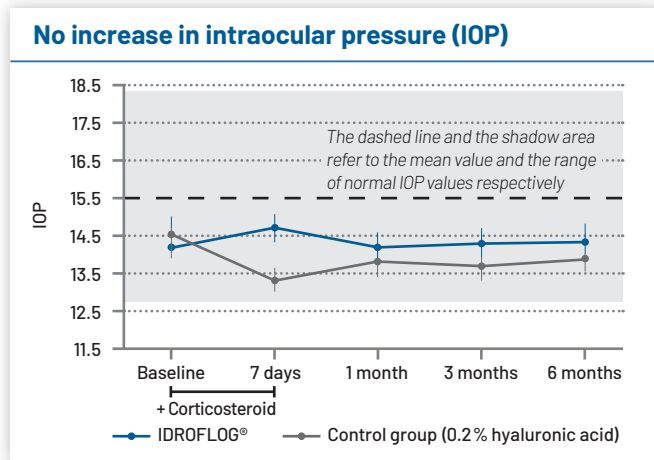

The intraocular pressure remained stable in both treatment groups. IDROFLOG® treatment did not result in an increase in intraocular pressure over the course of 6 months. There was also no significant difference between the two groups. Fig. modified after: Rolando M et al.³

IDROFLOG®:
available soon



- ▶ **No adverse events during the treatment period of 6 months reported³**
- ▶ **No increase in IOP during the treatment period of 6 months³**
- ▶ **IDROFLOG® will be available in 15 Single doses of 0.5 ml**

Product profile overview

IDROFLOG® is a medical device for the treatment of dry eye

Composition

- 0.2% hyaluronic acid (2 mg/ml sodium hyaluronate)
- 0.001% hydrocortisone (10 µg/ml hydrocortisone sodium phosphate)
- With citrate
- Without preservatives

Dosage/application

- 1-2 drops 2-4 times a day in the eye to be treated
- Total duration of treatment: 2-6 months
- IDROFLOG® may only be used under medical supervision
- Can be used together with all types of contact lenses

Packaging sizes

15 Single doses of 0.5 ml



¹ Craig JP et al. TFOS DEWS II Report Executive Summary. Ocul Surf. 2017;15(4):802-812.

² Rolando M, Barabino S. The Subtle Role of Para-inflammation in Modulating the Progression of Dry Eye Disease. Ocul Immunol Inflamm. 2021;29(4):811-816.

³ Rolando M et al. Long-Term Activity and Safety of a Low-Dose Hydrocortisone Tear Substitute in Patients with Dry Eye Disease. Curr Eye Res. 2023;48(9):799-804.

** Figure modified after ¹ and ².