
# ENDORET®

## MODEL SYSTEM V

MANUAL OF USE AND MAINTENANCE  
FOR THE PRGF SYSTEM CENTRIFUGE TO OBTAIN  
PLASMA RICH IN GROWTH FACTORS

<b>en</b>	ENDORET® MODEL SYSTEM V .....	(Page 2)
<b>es</b>	ENDORET® MODELO SYSTEM V.....	(Página 11)
<b>de</b>	ENDORET® MODELL SYSTEM V .....	(Seite 20)
<b>fr</b>	ENDORET® MODÈLE SYSTEM V .....	(Page 29)
<b>it</b>	ENDORET® MODELLO SYSTEM V .....	(Página 38)
<b>pt</b>	ENDORET® MODELO SYSTEM V.....	(Página 47)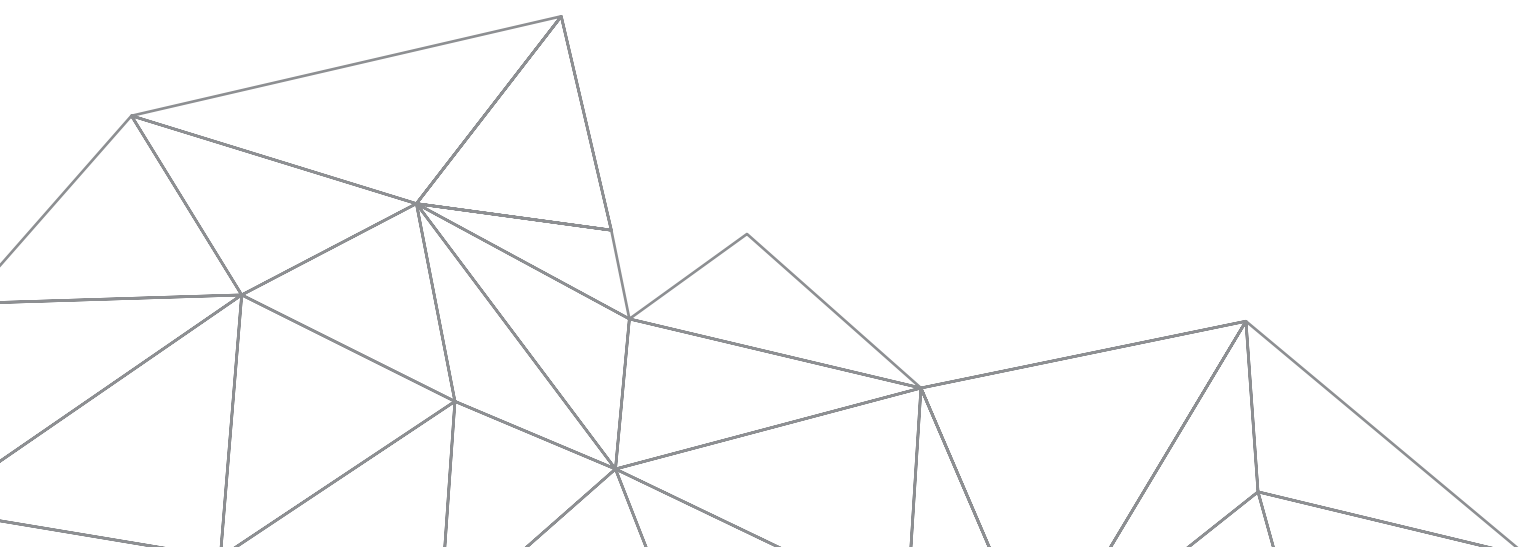

# ENDORET®

---

## MODEL SYSTEM V

### INTRODUCTION

#### PRECAUTIONS:

**To be able to maintain the operator's safety and ensure the centrifuge works perfectly, carefully follow the notes below, as well as all the warnings in this manual.**

BIOLOGICAL RISK

DANGER

The PRGF SYSTEM centrifuge model SYSTEM V is designed for use with the Technique of Obtaining Plasma Rich in Growth Factors, therefore its use entails the handling of blood fractions.

If you have to process blood from patients with infectious pathologies, you must be informed of the possible contamination and the danger that handling it entails.

Take the necessary precautions and use suitable decontaminating products, to protect yourself from contamination.

For greater safety, the centrifuge has a biological seal device, made up of the tube holder assembly of the hermetically sealed rotor, which prevents leakage of material and the dispersion of particles and aerosols.

The biological seals and their components are intended to be part of a biosafety system, as specified in the national and international biosafety indications. An important guide to biosafety can be obtained in the Laboratory Biosafety Manual, edited by the World Health Organisation. It is also necessary to comply with the current regulations related to exposure to biological agents.

Do not exceed the maximum values of the rotor, as this could lead to its failure.

Do not use the centrifuge if the rotor is not balanced. Ensure the rotor is loaded symmetrically, according to the specifications in this manual in point 3.2.1. Loading the

tube holder incorrectly can damage the motor assembly of the centrifuge or cause other damage.

Make sure you always place the safety seals on the tubes before starting centrifugation.

Do not lean on the unit or place objects on it while it is functioning.

If the centrifuge is used in a way other than that specified by BTI, its protection cannot be guaranteed, and it may be damaged.

**CAUTION:** *Children, pregnancy and lactation: There are no additional preventive measures other than those generally considered for PRGF-ENDORET® technology. However, these particular situations should be considered by the clinician before its application.*

#### SECURITY:

This unit, like any electronic equipment, is a potential source of electromagnetic radiation. It meets the requirements for electromagnetic compatibility, emission and immunity for electrical equipment for measurement, control and use in laboratories. (IEC 61326-1).

In any event, you must evaluate the electromagnetic environment before starting up the unit.

**CAUTION:** *Do not start up the centrifuge near sources of strong electromagnetic radiation (for example, unshielded intentional RF sources), as they may interfere with the proper operation of the centrifuge.*

Check the centrifuge's power supply requirements, so that it is not connected to an unsuitable source.

Use only the rotor and tube holder contained in the centrifuge.

For better conservation of the centrifuge, carefully follow the instructions in this manual.

**SYMBOLS:**

The symbols that figure on the unit are:

	Danger (see the documentation attached).
	Biohazard
	Danger, risk of electric shock.
	Danger, manual opening of lid
<b>Rx only</b>	The sale of this device is restricted solely to licensed health professionals
	Compliance with directive 93/42/EEC on medical devices

The symbols **DANGER, CAUTION AND BIOHAZARD** in this manual are used to emphasise critical and important instructions.

 **DANGER:** This symbol, accompanied by the word **DANGER**, indicates to the operator that this method of use may cause physical harm.

 **CAUTION:** This symbol, accompanied by the word **CAUTION**, indicates to the operator this method of use could damage the centrifuge.

 **BIOHAZARD:** This symbol indicates to the operator that using this unit with blood from patients with infectious pathologies can their physical integrity.

**SECTION I: INSTALLATION**


Instructions for correct installation of the centrifuge at its workstation.

**LOCATION:**

The surface where the centrifuge is going to be placed must be solid enough to avoid vibrations, taking into account that the weight of the centrifuge is 16.62 kg. Leave a safety distance of at least 30 cm at the front and 30 cm at the sides and rear. Within this safety distance there must not be any devices, walls, panels or any other objects or obstacles. Thus the operator's safety and good air circulation can be ensured. The table or worktop on which the centrifuge is placed must have access to a mains socket, at a maximum distance of 1 m. This mains supply must have the characteristics required for the centrifuge to work correctly. See section. "2.2 Technical characteristics".

**POWER SUPPLY:**

The mains connector is located at the rear, and comprises the master switch and the fuse box. Before connecting the mains cable to the unit, check the power supply values (voltage, frequency, etc.) are correct. You can find these values on the characteristics label next to the mains connector or in section 2.2. The power supply must be earthed and the centrifuge must be connected to this.

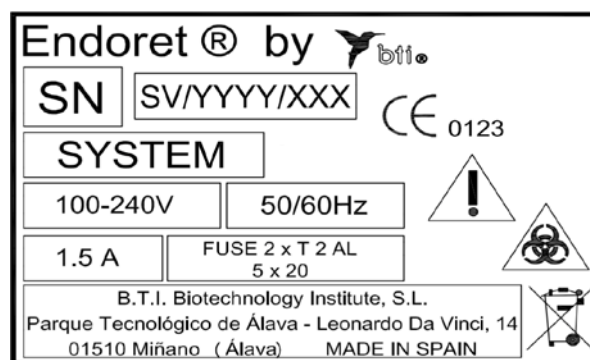**INSPECTION/INSTALLATION:**


Inspect the overall condition of the packaging, if you observe any deterioration (it is wet or broken) contact your distributor.

Once you have opened the packaging, remove the mains cable, the centrifuge, a bag with the user manual, the declaration of conformity and a couple of fuses.

 **DANGER:** Take care when transporting the centrifuge, as it weighs 16.62 kg. Incorrect handling during transport could physically harm the person transporting it and/or damage the unit.

 **CAUTION:** To transport the centrifuge, pick it up by the lower part of the casing. Holding the centrifuge by other part could damage some components of the unit.

- Once the centrifuge has been unpacked, place in its workplace. Ensure the safety distance are suitable in accordance with section 1.1 and that the legs of the centrifuge are all resting on the work surface. The thread of the legs must protrude as little as possible from the base and the nut must be screwed upward, next to the base, the rubber must not be bent. See Photo.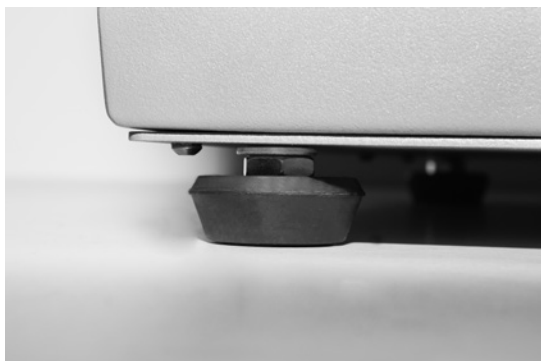

- Check that the master switch is in the "0" position (off).
- Connect the centrifuge making sure that the mains supply is as specified on the characteristics label in section 1.2 or 2.2.
- Press the master switch, press the ON/OFF icon when the touchscreen is enabled, and press the icon to open the door. Inside there should be 4 protected tube holders, with their bioseals and lids corresponding. Numbered correspondingly.
- Once they have been unpacked and placed in their position in the rotor, close the door.

The centrifuge is now installed.  
If any material is missing, it is not as specified or is defective, contact your distributor.

## SECTION II: DESCRIPTION

This section describes the technical and functional characteristics.

### FUNCTIONAL CHARACTERISTICS

The centrifuge is a robust product. Its exterior frame and its centrifugation chamber are made of enamelled steel. The inner parts are zinc-coated or are made of aluminium. The fasteners are stainless steel. This makes them highly resistant to corrosion and facilitates their cleaning.

The anodised rotor and tube holder offer a high resistance to corrosion and scratching, in addition to facilitating its cleaning. Their design makes it possible to achieve suitable speeds, maintaining its safety.

Using the centrifuge correctly, none of the materials that comprise the centrifuge will reach extremes of fatigue.

Equipped with a display and a touchpad to control the following parameters:

- switching on and off
- opening the cover
- start and emergency stop
- impossible start up the rotor with the cover open or not properly closed
- impossible open the cover if the rotor not is fully stopped
- automatic stop of the rotor in the event of excessive imbalance of the load, with acoustic warning
- automatic stop of the rotor in the event of emergency opening, with acoustic warning
- energy saving system in which after 5 minutes without use, the screen turns off
- warning of unbalanced rotor
- warning of motor fault

The display, the touchpad and the acoustic warning indicate the operational process it is in at all times.

The rotation speed is limited to 1902 rpm.

### TECHNICAL CHARACTERISTICS\*

Benchtop centrifuge:

- Power supply (**)	100 - 240 V
- Maximum frequency	50/60 Hz
- Max. power consumption	100 – 250 W
- Current	1.5 A
- Fuse (type T)	2 x T 2A L (5 x 20 mm)
- Maximum capacity	16x9 ml
- Max. centrifugal force	1902 rpm
- Max. time	8 min.
- Class (according to CISPR 11) (***)	
- Max. noise level	< 85 dB
- Program memories	1 factory-set program
- Temperature margin	2°C - 40°C
- Relative humidity	<75%
- Dimensions: (W/D/H)	360x455x239 mm
- Weight	16.62 kg.
- Warranty	24 (twenty-four months)

Transport and storage conditions:

- Humidity	< 90%
- Temperature	-18°C to 40°C

\* These technical characteristics are for:

- Ambient temperatures between 2°C and 40°C.
- Grid margin  $\pm 10\%$ .
- Relative humidity < 75%
- For installation category II b.
- Pollution class I

- Indoor use
- Altitude up to 2000m

\*\*The grid must be earthed and the centrifuge must be connected to it.

\*\*\*It is authorised for use in domestic establishments, under the jurisdiction of a medical professional.

**ACCESSORIES:**

No accessories have been designed for the centrifuge.

**CAUTION:** Use only the rotor and the tube holder contained in the centrifuge. To guarantee the safety of the user, the rotation frequency has been limited to 1902 rpm.

**SPARE PARTS:**

To be able to supply original spare parts, you must indicate the code or description of the component and the serial no. of the centrifuge.

**CAUTION:** BTI shall not be held responsible for damage that may be caused by non-original spare parts. Only use original spare parts.

**RS2.5 PORT:**


The unit has an RS232 interface on the rear panel of the unit for exclusive use by BTI technical staff. Do not connect any other unit to this Port.

**SECTION III: OPERATION**

To be able to begin work with the centrifuge, first you must familiarise yourself with it and with each one of its components, in particular the control panel, from which you can access all the menu options.

**CONTROL PANEL:**

This image shows the display and the touchpad.

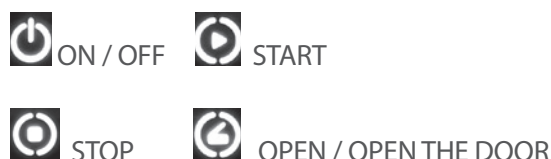**ROTOR**

**Loading and balancing the rotor:**

Check that nothing has fallen into the chamber (clips, stoppers, etc.) before loading the rotor. Once you have made these checks, load the rotor and balance it correctly, taking into account the following instructions:

- Use BTI tubes with the same volume of 9 ml. NEVER use the centrifuge with other tubes even if they are of the same volume.
- The tubes must hold their complete volume (they must be full). Place the tubes in opposing positions inside the tube holder, maintaining a geometrically symmetrical layout.
- Use a tube filled with water if you are one tube short to complete a geometrically symmetrical layout. Symmetrical loading is achieved by tracing an imaginary line between any tube and the one opposite, passing through the axis of the rotor. See Photo.
- Be sure to correctly close the lids of the tube holders before starting centrifugation, as these may cause it to be off balance.

**If you do not follow these rules and fail to load the centrifuge correctly, there will be excessive vibration, which will cancel the operation command. The corresponding message will appear on the display and/or the result of the centrifugation will not be as expected.**

 **CAUTION:** Do not centrifuge with tubes with different volumes. Use BTI 9 ml tubes.

Do not use the centrifuge if the rotor is not balanced. Ensure the rotor is loaded symmetrically. Poor loading in the rotor can damage the centrifuge.

Incorrect positioning of the tube holder of the rotor may cause the rotor to be off balance.

Be sure to close correctly the sealing lids of the tube holder before starting the centrifuged, as this may cause a lack of balance.

#### INSPECTION AND CLEANING:

It is necessary to inspect and clean both the rotor and the tube holder when necessary, so as not to accumulate substances that may degrade the material with which it is built.

The cleaning instructions are described in section 4.2 of this manual.

 **DANGER:** Inspecting the rotor, to verify the absence of deterioration of the material, impacts, or fissures, is important, to be able to correct any possible faults in the unit. Remember that the worst accident that can be anticipated would be caused by the head breaking. Apply the same criteria to the tube holder.

#### OPERATING THE CENTRIFUGE:

##### Start-up:

Start up the centrifuge as follows:

- Connect the machine to the grid. Then press the network filter switch, to place it in the on position.
- Touch the touchpad so that all the options remain fixed.

- To switch it on, press the ON/OFF icon, which at the same time activates the software of the machine.
- If you wish to turn off the machine, press the ON/OFF icon.
- To open the cover, press the open cover icon and access the tube holder.
- Insert the tubes to centrifuge in their corresponding tube holders, after unscrewing their lids, taking into account that the machine must be balanced. Close the tube holder and the cover of the centrifuge. If you have made a mistake or you have left out a step, simply press the button to open the cover and go back a step.
- Close the cover to be able to start up the cycle. Verify that the cover is properly closed. If it is open, the display will show a message indicating that the cover is open or not properly closed.
- Press the start icon to begin the centrifuge cycle. The display will indicate the acceleration process, until the programmed speed is reached. The time will count down from the value selected.
- To stop it manually, press the emergency stop indicator the work cycle will be automatically interrupted. The interrupted cycle message will then appear.
- After the time selected in the work cycle has transpired, the centrifuge will stop automatically, going into the braking process.
- After the shutdown, the rotor speed will slow to zero. Immediately afterwards, an acoustic warning will indicate that the work cycle has ended.
- You can then remove the tubes from the chamber by pressing the open cover icon. This opens the cover of the centrifuge so you can access the tube holder and unscrew the lid of each of the loaded tube holders.

 **DANGER:** The seal must not be handled by anybody other than BTI technical staff. Do not try to force the door open if the lock has not yet been released. Do not try to start a cycle by placing a weight on the door.

#### OPERATIONAL PROCEDURE:

The steps to take for this technique are the following:

##### Drawing the blood

The patient to be treated will have a volume of between 9 and 144 ml drawn depending on the procedure to be performed.

These volumes are collected in 9 ml sterile blood extraction tubes, containing sodium citrate at 3.8% as an anti-coagulant.

## Centrifuging the blood

To centrifuge the blood fractions, use the same citrate tubes used to draw the blood.

Follow the instructions in section 3.2.1 to load and balance the rotor correctly.

Start up the centrifuge by following the instructions in section 3.3.1 Start-up.

Once the centrifugation process has ended, remove the tubes and place them in a rack for subsequent processing according to the instructions for use of the BTI Tubes or any other PRGF- ENDORET® technology available.

## Work cycles:

The work cycles are factory set so you do not have to select the parameters in each case; you do not even have to save them in the memory.

## Low-power mode:

This operating mode automatically switches of the machine to save energy. After 5 minutes without operation, such as pressing the screen, closing the cover, etc, the screen will turn itself off, and the centrifuge will remain on stand-by. To reset the centrifuge, simply press any icon on the keypad.

## OPENING THE COVER MANUALLY

This may only be used when unpacking or transporting the centrifuge, in the event of a power cut or failure of the automatic opening system. On the left side, next to the caution label, there is a recessed plastic screw that, when extracted, activates the emergency or manual opening system. Unscrew it with a Phillips screwdriver.

 **DANGER:** *In the event of a power cut, use this emergency or manual opening system. To use it, you must be sure the rotor has stopped completely. Never use it as a normal method of opening.*

## CENTRIFUGING DANGEROUS MATERIALS:

This centrifuge is designed with an air circulation system. This system consists of evacuating air absorbed inside the centrifugation chamber out of the unit. As a consequence of this the air emitted could be hazardous if the substance centrifuged is dangerous.

When maintenance or repairs must be performed on the centrifuge or any of its accessories, they must be cleaned and decontaminated, as described in section 4.2 of this manual.

 BIOLOGICAL RISK  DANGER

If you have to process blood from patients with infectious pathologies, you must be informed of the possible contamination and the danger that handling it entails.

Take the necessary precautions and use suitable decontaminating products, to protect yourself from contamination.

## SECTION IV: MAINTENANCE

This chapter describes the routine maintenance procedures, both mechanical and electrical, that must be carried out periodically, to maintain the safety of the unit. It is your responsibility to carry out the following procedures when necessary, and keep the centrifuge in good working condition to ensure the accuracy of the results. If you need the electrical and/or electronic diagrams for the correct maintenance of the unit or if, in future, you need our technical service, contact BTI.

## TECHNICAL ASSISTANCE

This type of operation must always be carried out by specialised personnel authorised by the BTI Technical Service. If the centrifuge has been used with pathogenic materials and requires revision by the technical service, the user must complete the procedure for cleaning to ensure the safety of all staff before performing any repair or maintenance work. In the event that the technical service uses

any of these substances, it will also be obliged to carry out the following cleaning procedure: clean the centrifuge of all incrustated material and decontaminate it according to section 4.2 of this manual.

#### ANOMALIES:

- The centrifuge does not start up and indicates that the cover is open:  
Check that it is properly closed. If it is not, unlock it using the emergency opening system, and verify that the cover opens and closes correctly.
- The unit is connected to the grid, the master switch is connected and the mains connection indicator does not light up: (2):  
Replace the fuses according to section 4.3.

#### INSPECTION AND CLEANING

It is important to carry out a general monthly inspection of all the labels of the centrifuge and, if necessary, clean them so they are legible.

Before using any cleaning or decontamination agent other than those recommended in the manual, check with BTI that the methods proposed are not going to damage the unit.

#### EXTERIOR OF THE CENTRIFUGE

Clean the centrifuge with a damp cloth in a non-alkaline detergent. Rub gently until clean, rinse the cloth with water and then wipe again. Dry with an absorbent cloth. Do not use abrasive products.

#### CENTRIFUGATION CHAMBER

The centrifugation chamber is manufactured in steel with a surface treatment, which can be cleaned easily. This includes the external cover, so the cleaning and decontamination process must contemplate this component. Always keep it clean and dry. To do so, remove the tube holder and wash the chamber with a cloth soaked in neutral soap product, and disinfect as below.

#### ROTOR AND TUBE HOLDER

The rotor and the tube holder are made of aluminium alloy and hydrochloric products are harmful, and may damage them. If it is necessary to disinfect them, ensure you rinse them well with water to remove any residue of these types of products. The rotor can be cleaned using a soft cloth soaked in a soapy product to remove all the encrusted substances. Rinse and dry with a soft, absorbent cloth

 **DANGER:** *Inspecting the rotor is important to be able to neutralise any possible failures of the unit. Remember that the worst accidents anticipated would be caused by this breaking.*

#### BIOSEALS

It is advisable to replace the bioseals (tube holder gaskets) at least once every 3 years, to guarantee the integrity of the microbial barrier. Inspect them and replace them then. In the event of frequent use or malfunction, carry out this verification more frequently.

Contact your BTI distributor to guarantee a correct replacement and installation of the accessory replaced.

#### Disinfection:

- If disinfection is required, make sure first to clean properly and rinse it well with water to eliminate any type of dirt and then proceed to disinfect the unit by using a 70% Ethanol solution.
- Rub gently all the affected surfaces with a 70% Ethanol solution dampened cloth.
- Do not dry the unit with a cloth, wait until the ethanol solution evaporates by itself.

#### CHANGING FUSES

The fuses are in the base of centrifuge connector. To replace them, first remove the mains cable and use a screwdriver to lever the fuse box cover. Extract and change the deteriorated fuses for new ones of the same amperage, which will be indicated on the characteristics label of the centrifuge or in section 2.3 of this manual.

 **DANGER:** *Before proceeding with this maintenance process it is essential to disconnect the mains cable from the power supply.*

#### CHANGING THE ROTOR

The rotor may only be replaced by BTI staff or personnel authorised by BTI. Rotor models other than the one included in the machine may not be used. BTI does not supply this component to users.

 **DANGER:** *The rotor cannot be replaced with any other that is not this model. Any work on this component may only be carried out by BTI staff or personnel authorised by BTI.*

## DISPOSING OF THE USED PRODUCT

This symbol on the product, its accessories or the packaging indicates that the product must not be treated as domestic waste. Dispose of the unit at your nearest collection point for the recycling of electrical and electronic waste. In the European Union and other European countries there are different collection systems for used electrical and electronic products. Ensuring this product is disposed of correctly will help avoid potential risks for the environment and the health of people that could occur if the product is not disposed of properly. Recycling of materials helps preserve natural resources. Therefore, do not dispose of used electrical and electronic equipment or batteries in the domestic waste. For more detailed information on the recycling of this product, contact your municipal office, the domestic waste collection service or the establishment where you acquired the product.

## APPENDICES

### A.1 Warranty

BTI guarantees this unit against all manufacturing defects for a period of 2 years, from the purchase date, in the following cases:

\*Covering any manufacturing defect, including the labour necessary to locate and replace of the defective parts in our Technical Service.

\*This guarantee DOES NOT COVER faults that the BTI Technical Service deems to have been caused by incorrect installation, mistreatment, improper use or manipulation by people other than the BTI Technical Service.

\*Any acknowledgement by BTI of compensation due to direct or indirect harm of any nature suffered by people or objects is expressly excluded.

\*Any unit that has had the characteristics label removed or altered is not covered by this warranty.

### A.2 Manufacturer details

Name: B.T.I. BIOTECHNOLOGY INSTITUTE S.L.  
 Address: Parque Tecnológico de Alava  
 Leonardo da Vinci 14  
 01510 MIÑANO  
 ALAVA (SPAIN)  
 Telephone: +00 945297030  
 Fax: +00 945297031

### A.3 Configuring the language

1. Press the master switch or the network filter, setting it to position I.

2. Press the icon  to set the touchpad.

3. Press the icon  again to switch on the machine.

4. At the end of the countdown, image 1 will appear and with the icon  you can scroll through the languages until you reach the one you want. Image 2. Press the icon  to confirm it.

Image 1

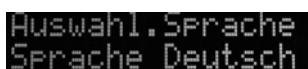
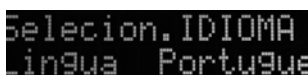
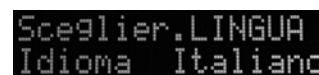
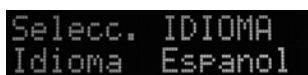
Image 2

5. It will ask you twice more image 3 and if it is the desired language press the icon  to save it. Image 4.

Image 3

Image 4

## IN CASE OF ERROR

6. The two times it is pressed before it is saved you can press the icon  for go back and correct it.

7. If you have selected the wrong language, turn the unit off and on again.

8. During the countdown, press the icon  for two seconds. Go back to step 3.

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