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
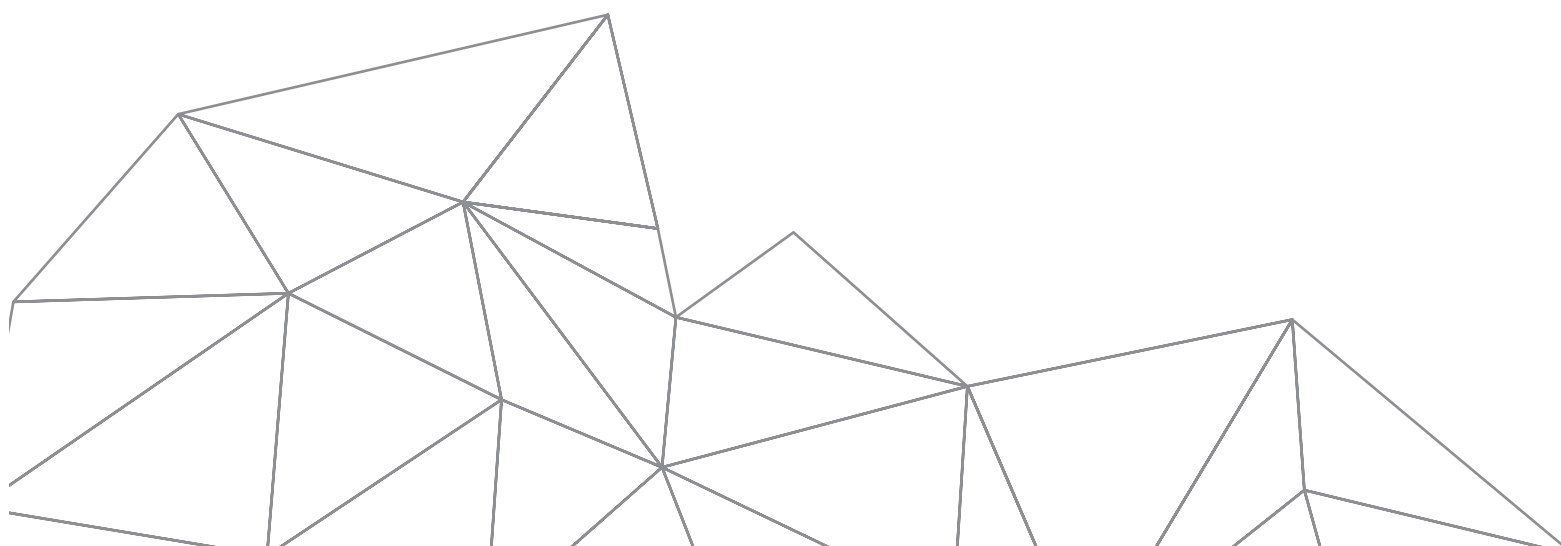
ENDORET®

OPHTHALMOLOGY KIT and PRGF EYE DROPS: INFORMATION FOR THE PATIENT

	en	ENDORET® OPTHALMOLOGY KIT and PRGF EYE DROPS: INFORMATION FOR THE PATIENT	(Page 2)
	es	KIT ENDORET® OFTALMOLOGÍA y COLIRIO PRGF: INFORMACIÓN PARA EL PACIENTE	(Página 3)
	de	ENDORET® AUGENHEILKUNDE-SET und PRGF AUGENTROPFEN: INFORMATIONEN FÜR DEN PATIENTEN	(Seite 4)
	fr	KIT D'OPHTALMOLOGIE ENDORET® et GOUTTES OPHTALMIQUES PRGF : INFORMATIONS POUR LE PATIENT	(Page 5)
	it	ENDORET® OPTHALMOLOGY KIT e PRGF COLLIRIO: INFORMAZIONI PER IL PAZIENTE	(Página 6)
	pt	KIT DE OFTALMOLOGIA ENDORET® e COLÍRIO PRGF: INFORMAÇÕES PARA O PACIENTE	(Pagina 7)
	cs	OFTALMOLOGICKÁ SADA ENDORET® a OČNÍ KAPKY PRGF: INFORMACE PRO PACIENTA	(Strana 8)
	sk	ENDORET® OFTALMOLOGICKÁ SÚPRAVA a PRGF OČNÉ KVAPKY: INFORMÁCIE PRE PACIENTA	(Strana 9)
	no	ENDORET® OFTALMOLOGISETT og PRGF ØYEDRÅPER: INFORMASJON TIL PASIENTEN	(Side 10)
	sv	ENDORET® OFTALMOLOGI KIT och PRGF-ÖGONDROPPAR: INFORMATION TILL PATIENTEN	(Sida 11)

ENDORET® OPHTHALMOLOGY KIT AND PRGF EYE DROPS: INFORMATION FOR THE PATIENT

PATIENT INFORMATION SHEET:

What it is

The PRGF eye drops obtained through the ENDORET® technique is a preparation of autologous eye drops, in other words, obtained using the patient's own blood, which prevents incompatibility problems. It contains components such as growth factors, vitamin A and fibronectin and has physiological characteristics like lacrimal fluid (tears).

Presentation and composition

A protective bag having inside a rack with 31/32 or 15/16 droppers, depending on whether microbiological control has been carried out at the discretion of the specialist prescribing, each containing 0.6 ml of PRGF eye drops obtained through the ENDORET® technique for ophthalmic use and one conservation container. The person responsible for the preparation of this ophthalmic solution is the specialist prescribing.

How to transport and store it

After receiving from the doctor, you should store the PRGF eye drops as soon as possible in a freezer at temperatures below -15 °C. Until then and during transportation from the medical office to your home, PRGF eye drops can be stored at room temperature (below 20-25 °C) when shipment does not exceed 3 days or refrigerated (2-8 °C) when delivery is expected to be between 3 to 7 days. BTI recommends its COOLBOXes (CBOX, CBOXM), which maintain the temperature between 2-8 °C. Immediately after arriving home, you must store the PRGF eye drops in a freezer at temperatures below -15 °C.

When to start the treatment

Do not start the treatment until you receive confirmation from your Ophthalmology Centre or Clinic. This notification could take about seven days from the day the doctor gave you the PRGF eye drops.

If results are satisfactory, you will be notified to start the treatment. If not, you will be notified to discard the PRGF eye drops. You can dispose the droppers as conventional organic waste.

How to start the treatment

After receiving confirmation for starting the treatment, you must transfer a dropper from the freezer to the refrigerator at least 8 hours before use (usually before bedtime); allowing PRGF eye drops to thaw.

To do this, open the protective bag by raising the yellow seals and rolling out the bag. Take out the rack and transfer one of the droppers into the conservation container. Check the conservation container is properly closed before placing it in the refrigerator.

Place the rack inside the protective bag, close it by rolling in the open end and folding back the yellow seals that are protruding from the protective bag. Return the rack to the freezer.

How to use the PRGF eye drops

For a correct administration, wash your hands before opening the dropper and make sure that the end of the dropper does not touch any surface.

Then pull down the lower lid of the affected eye and carefully apply the indicated number of drops in the area of the lesion.

After each use, close the dropper and place it inside the conservation container. Make sure to close it.

During the treatment, store the dropper in use preferentially refrigerated (2-8°C) although, if it is not possible, it can also be kept at room temperature (below 20-25°C).

 Once a dropper is first opened it can be used for 72 hours.

Dosage

Carefully follow the guidelines indicated by your doctor. Do not modify the dosage yourself.

Continue the therapy for the whole treatment period indicated by your doctor.

To be filled in by the doctor:

Date of manufacture	
Treatment duration	
Dosage	
Observations	

Handling precautions

 Handle the droppers very carefully; do not, for any reason, press or force the vial while it is closed.

 During the application, do not let the dropper touch your eye or your fingers.

 Make sure you have completely closed the droppers after each use.

 During treatment, always keep the dropper in use inside the conservation container. Make sure the conservation container is completely closed.

NOTE: The devices should be accepted only if the packaging and labeling arrive intact. Contact your doctor if the package is opened or altered.

Disposal

Each dropper can be used for a maximum of 3 days. At the end of the 3rd day the dropper in use must be disposed as conventional organic waste. Do not dispose the conservation container.

To continue the treatment, transfer a new dropper from the freezer to the refrigerator to allow thawing.

Patient responsibilities and precautions

- Use these PRGF eye drops according to the instructions for use and precautions described; follow the guidelines established by your doctor and do not modify this dosage yourself.
- These PRGF eye drops were prepared using blood extracted from you, so they are exclusively for your use. NEVER treat other patients with your eye drops.
- Store the product in accordance with the instructions stipulated; not doing so may lead to the functional impairment of the preparation and risk of infection.
- The PRGF eye drops must be kept out of reach and sight of children.
- If you detect any type of adverse reaction or eye infection, contact a doctor immediately.

Meaning of the symbols used in this manual

	Caution
---	---------

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